Helios DRF

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**Helios DRF: lightness, compactness and
reduced dimensions.**

Each movement has been optimized to facilitate the use by the operator and to ensure maximum patient safety, preventing his movement during the examination

The minimum height from ground of the patient tabletop is just 45 cm, the lowest of the category.

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X-ray tube head motorized rotation +/-180°

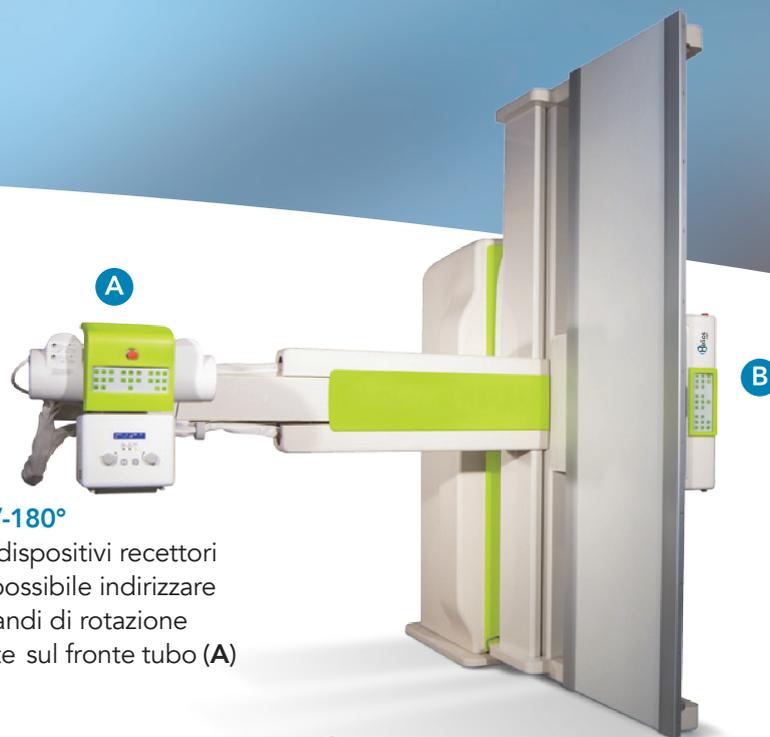
It allows the directing of the beam towards any external receptor devices. As an example, when the table is tilted at +90°, the x-ray tube can be directed towards a stretcher or a wall stand. It is activated from remote or from the two control boards placed on DRF (B) and tube front (A).

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Focal distance up to 180cm (optional 200cm)

Adjustable focus to film distance from 105 to 180cm. Helios DRF allows chest examinations directly on the table, be it in horizontal or vertical position.

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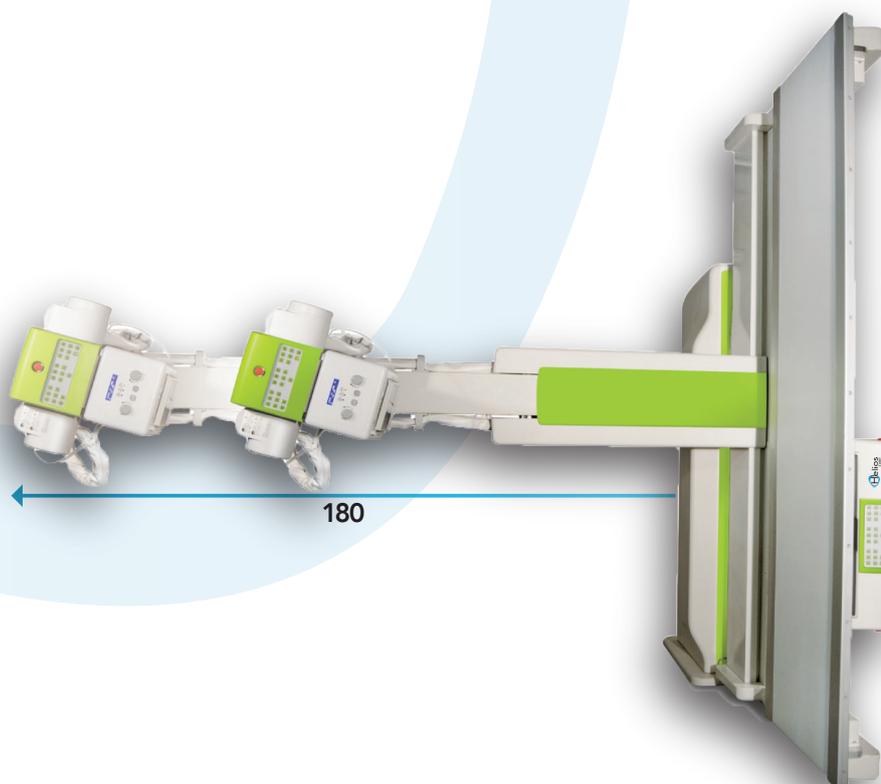
Retractable grids

The table is supplied with one or more retractable grids having different focal distances. Two rails under the flat panel holder group allow quick and safe parking. The operator can select among the following modes:

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Elevated Coverage

The large dimensions of the tabletop 250x80cm (optional 250x80cm) linked to the longitudinal excursion of the X-ray tube column and detector allows a patient total scan of 210cm. In this case there is no need to reposition the patient. **It is a perfect solution when the patient is not collaborating.**

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Tomography

The tube motorized rotation is suitable to the execution of tomography and oblique projections without the mechanical bar support. The tomography angles (for a total of 5) can be chosen by the customer at the order.

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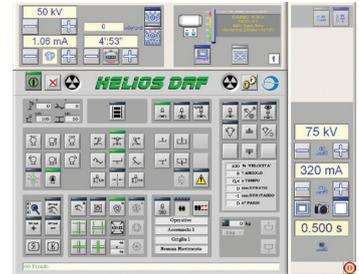
Integrated remote control console

Remote control console with 19" touch screen monitor. It allows all table movements, exam preparation and generator settings. The console includes an additional set of 4 joysticks to control the main movements like tilting, longitudinal travel, column displacement, etc.

Best ergonomics: all tables controls are further duplicated on the two control boards placed on flat panel and X-ray tube front.

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TLK Movement

When the TLK option is chosen, the tilting movement of the patient tabletop clears the exam area from itself so allowing contact radiographies as well as exams on patients on stretchers (radiography and fluoroscopy).

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Abdominal compressor

The abdominal compressor with automatic parking is optionally available. It is controlled from console with display of the compression force and from the table control boards. Retrofit available.

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"Two in One" full digital acquisition system

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Complete dynamic procedures and high resolution radiographic acquisitions for a full range of fluoroscopic, angiographic and radiographic applications.

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*Opzionale

Dynamic Imaging for fluoroscopic and angiographic applications:

- Gastrointestinal examinations
- Urogenital tract examinations
- Endoscopy
- Arthrography
- Venography
- Lymphography
- Myelography
- Step Angio
- Digital angiography*
- Digital subtracted angiography*

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*Opzionale

Static Imaging for radiographic examinations

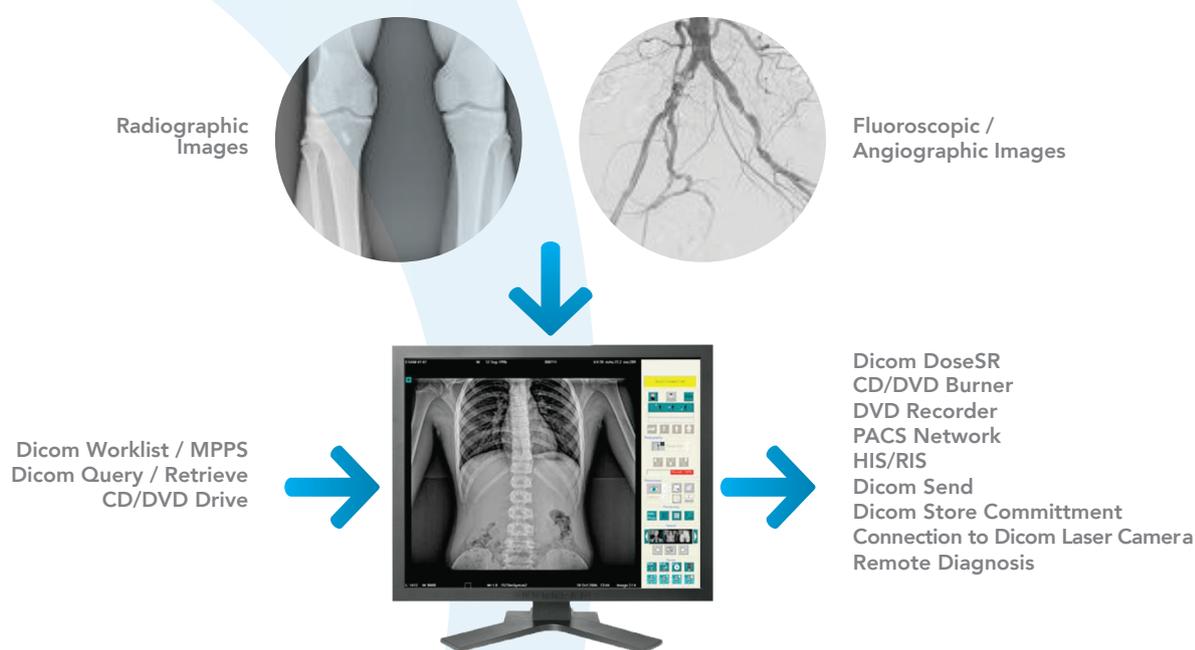
- Skeletal radiography
 - Skull
 - Thorax
 - Abdomen
 - Vertebral column
 - Pelvis
 - Upper and lower extremities
 - Tomography
 - Long image display or stitching *
 - Tomosynthesis

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Enhanced workflow. Total connectivity

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"2 in 1" efficiency

With Helios DRF the time needed to perform mixed fluoroscopy and radiography examinations is considerably reduced, using now the same system with direct to digital technique.

Reduce time

The use of cassettes and film developers tends to disuse. With Helios DRF the images are accessed in real time, shortening examination times for the operator and the patient.

Optimized connectivity

Thanks to its Dicom 3.0 capabilities, the system can be seamlessly integrated with existing HIS/RIS and PACS. If the objective is a filmless environment, it is possible to create CD's and DVD's directly from console with the integrated CD/DVD writer, or alternatively to send to print only the most significant images.

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Software grid

Through the Software Grid, it is possible to take grid-less examination and, consequently, to reduce patient dose. An algorithm simulates the physical grid resulting into high resolution images with no artifacts. It is mainly recommended for thorax or abdomen exams where the x-ray diffusion is generally very high.

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Dual Energy

Through the dual energy (DE) acquisition, the software generates three different images. The first one shows the soft tissues, the second image shows the bones and the third is an overlap of the first two images.

This is an ideal technique for chest x-rays as it helps in detecting pulmonary nodules. Moreover, dosage in DE is by far lower than in dedicated CT scans, which can be considered as a second level analysis.

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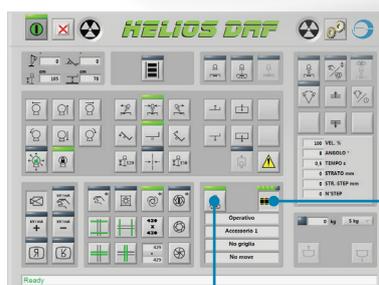
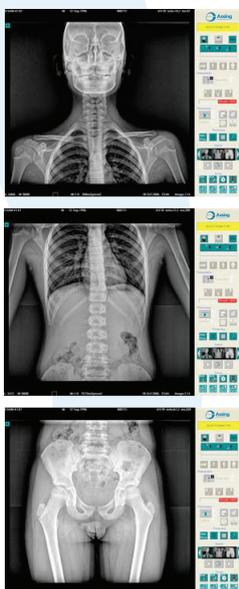
Grid suppression

New suppression algorithm that eliminates the artifacts without compromising image details.

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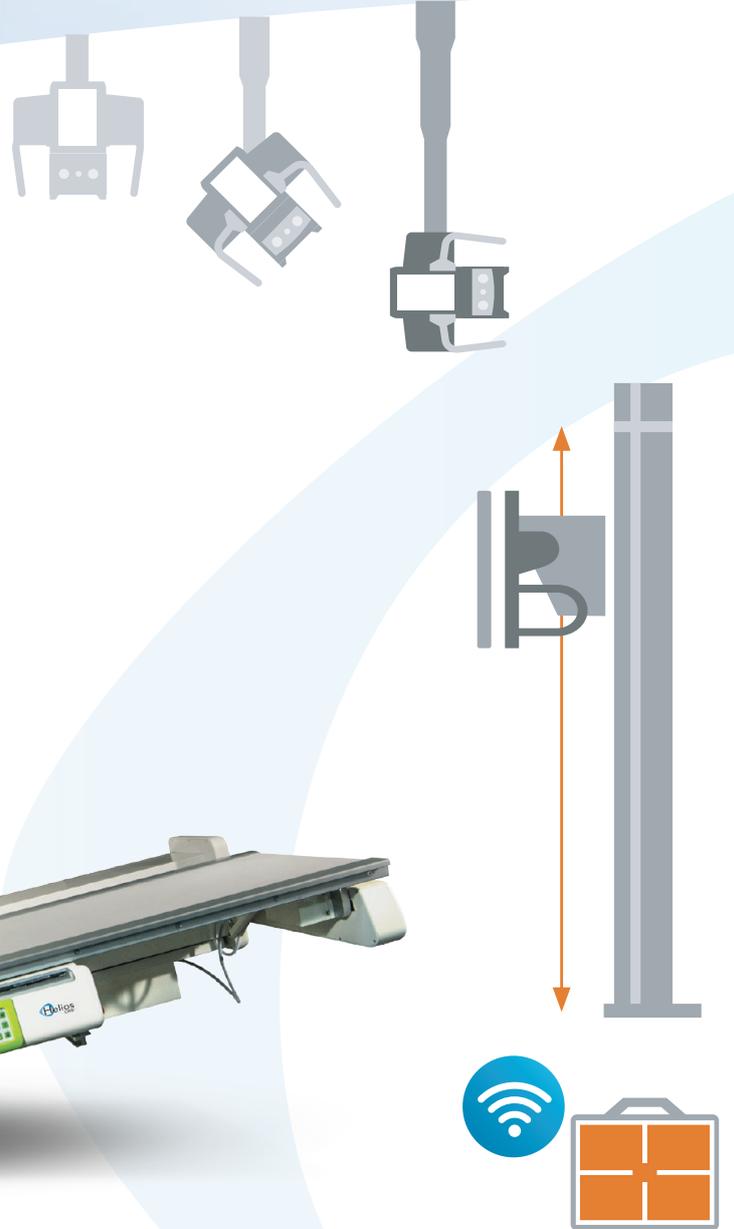
The stitching function consists of an automatic reconstruction of an X-ray image starting from a series of images acquired at fixed frequency during the scanning of the patient. The image is reconstructed, keeping all original pixels, and can be viewed on the monitor, processed, printed or sent to the network. This function is usually used for the scan of spine and lower arts. Using the "stitching" command, the system automatically generates a sequence of acquisitions according to the number of frames selected by the operator (2 for 70 cm, 3 for 100 cm, 4 for 130 cm length). Afterwards, the software links the images to get a single one shown in approximately 5 seconds.

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Helios DRF can be combined to the use of a tube stand, a vertical stand and a 35x43 or 43x43 cm cassette-sized wireless flat panel.

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With the use of a wireless portable detector and a ceiling tube stand the system achieves maximum flexibility making imaging possible virtually everywhere in the room, as conducting examinations on patients in wheelchairs or stretchers. Adding a vertical bucky stand would complete the spectrum of upright examinations such as lateral projections.

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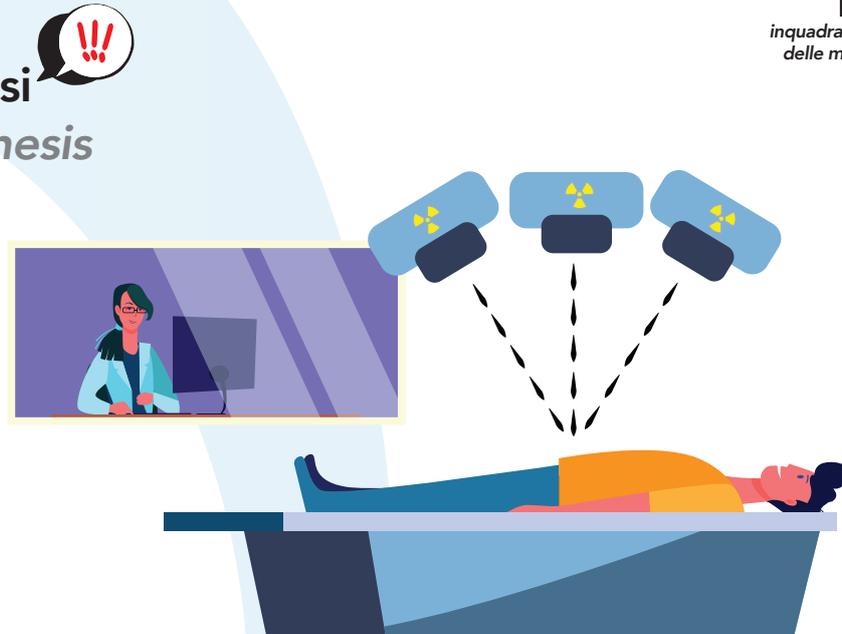
A single acquisition console is able to capture images coming from both dynamic flat panel and from the wireless one.

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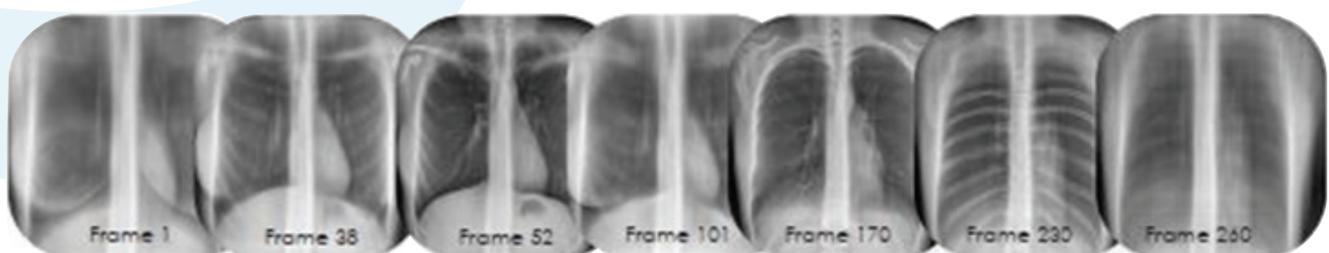
The Tomosynthesis is a technique which allows the reconstruction of volumetric images starting from a finite number of bi-dimensional projections taken at different tube angles. This is given by the separation of the acquisition process from the visualization one.

Among the advantages with respect to CT scan technique, it is well accepted by those patients affected by severe claustrophobia. Tomosynthesis has several areas of application including:

- Weight bearing views
- Knee pain
- Spontaneous Osteonecrosis
- Bone on bone
- Worn cartilage

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Assing has 3 branch offices in Italy and a network of authorized dealers in Italy and abroad.

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A good product is nothing without the right after-sales service.

Assing dedicated after sales service is our flagship. It is operating from our branch offices all over Italy and can constantly assist our customers.

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The after sales service can be contacted h24.

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ASSING SpA